A large, dynamic splash of water droplets is centered on the page, set against a solid blue background. The droplets are highly reflective, showing highlights and shadows that give them a three-dimensional appearance. The splash is oriented vertically, with the largest droplet at the top and smaller ones trailing downwards.

Cochlear™

Nucleus® Aquat+
Recipient Kit Instructions

Hear now. And always.

Cochlear®

Notes

A series of horizontal dotted lines for writing notes.

Contents

| | | |
|-----------|-------------------------------------------------------------|-----|
| EN | Cochlear™ Nucleus® Aqua+ Recipient Kit — Instructions | 2 |
| DA | Instruktioner til Cochlear™ Nucleus® Aqua+ sæt | 18 |
| FI | Cochlear™ Nucleus® Aqua+ -aloituspaketin käyttöohjeet | 34 |
| NO | Instruksjoner for Cochlear™ Nucleus® Aqua+-brukersett | 50 |
| SV | Instruktioner för Cochlear™ Nucleus® Aqua+ startkit | 66 |
| LT | „Cochlear™ Nucleus® Aqua+“ recipiento komplekto instrukcija | 82 |
| LV | Cochlear™ Nucleus® Aqua+ pacienta komplekta instrukcijas | 98 |
| ET | Toote Cochlear™ Nucleus® Aqua+ asenduskomplekt juhend | 114 |

Cochlear™ Nucleus® Aqua+ Recipient Kit

EN

Cochlear™ Nucleus®
Aqua+

Cochlear™ Nucleus®
Aqua+ Coil and magnet

Cochlear™ Nucleus®
Mic Lock-Stirrup
(optional)

Cochlear™ Nucleus®
Safety Line
(optional)

The **Cochlear™ Nucleus® Aqua+** is a re-usable sealable sleeve that, when used with the Aqua+ coil, keeps your sound processor dry during use in or around water (e.g. when swimming, surfing, or just having fun splashing around). It has a built-in ear hook and is worn behind the ear with your sound processor sealed inside.

The Cochlear™ Nucleus® Aqua+ is intended for use by recipients using the CP810, CP910, and CP920 sound processors with rechargeable batteries. It should only be used by recipients who can either remove the sound processor themselves if it's causing discomfort or who can indicate any discomfort to their parent or caregiver.

You must use rechargeable batteries for your sound processor to work inside Aqua+. Zinc air batteries will not work as they need air and there is no air inside Aqua+ when it's sealed.

The **Cochlear™ Nucleus® Aqua+ Coil** (with integrated coil cable) is used in place of your usual coil and coil cable while your sound processor is inside your Aqua+. It has a special plug that creates a seal when connected to your sound processor's coil socket through the coil plug hole in Aqua+. As with your usual coil, a coil magnet screws into the Aqua+ coil and holds it against your implant.

The **Cochlear™ Nucleus® Mic Lock-Stirrup** is a retention option to hold Aqua+ more securely behind your ear. The stirrup connects to the base of Aqua+ and the Mic Lock tube (when cut to size) passes under the ear lobe and connects to the Aqua+ earhook.

The **Cochlear™ Nucleus® Safety Line** is a retention option to help prevent accidental loss of your Aqua+ by securing it to your clothing with a snap-lock clip.

Remotes and wireless accessories can be used with Aqua+ if they are suitable for use in or around water.

Testing shows Aqua+ can be used for 2 hours and at depths of up to 3 metres.

Testing shows Aqua+ can be re-used up to 50 times, but this may vary depending on your particular use of it.

There have been no reported undesirable side effects from using Aqua+.

Instructions for Cochlear™ Nucleus® Aqua+

Before you get wet...

- 1 Carefully remove your usual earhook and unplug your usual coil from the sound processor (see your sound processor's user guide for specific instructions).

- 2 Insert your sound processor into Aqua+. You don't have to attach the Aqua+ earhook to your sound processor.

Don't forget you must be using rechargeable batteries for your sound processor to work inside Aqua+.

- 3 Plug the blue Aqua+ coil (with magnet) straight into your sound processor through the hole in Aqua+. **Do not twist.** The coil plug should feel securely connected.

- 4 Insert the plug into the base of Aqua+. To make a good seal, push the plug in as far as it will go (like a cork).

- 5 If you're worried about losing Aqua+ (e.g. in the surf), use the safety line to secure Aqua+ to your clothing.

- 6 If you're concerned about keeping Aqua+ on your ear, attach a Mic Lock-Stirrup (or another retention aid).

See 12 for instructions on cutting Mic Lock-Stirrup to fit your ear.

- 7 Finally, do the **Aqua+ Checklist** to make sure you're ready to get wet!

- 8 Have fun!

After getting wet... Care and storage

- 1 Rinse Aqua+ with clean water and dry with a soft cloth.
- 2 Unplug the blue Aqua+ coil. Pull the plug straight out; **do not twist**.

- 3 Open Aqua+ and remove your sound processor.

- 4 Re-attach your usual coil and earhook.

- 5 Place Aqua+ and the blue Aqua+ coil in your dry aid kit overnight.

- 6 Once they're thoroughly dry, store Aqua+, the blue Aqua+ coil and any accessories in your Cochlear™ Nucleus® Storage Case.

Instructions for Cochlear™ Nucleus® Aqua+ Coil

- 1 Insert the Cochlear™ Nucleus® CP900 Series Magnet into the hole in the blue Aqua+ coil. Turn clockwise until the magnet is set at a comfortable depth (check it against your implant).

Once fitted, leave the magnet in your blue Aqua+ coil.

Instructions for Cochlear™ Nucleus® Safety Line

- 1 Thread the loop end of the safety line through the Aqua+ ring.

- 2 Create a knot by passing the clip end of the safety line through the safety line loop.

- 3 Lift the lever to open and close the clip.

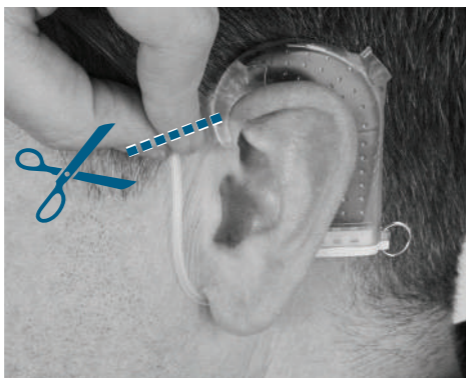
Instructions for Cochlear™ Nucleus® Mic Lock-Stirrup

Before using Mic Lock-Stirrup for the first time, cut it to the right length to suit your ear.

- 1 Snap the stirrup into the dimples at the base of Aqua+.

- 2 Find the right length of tubing to hold Aqua+ in place comfortably behind your ear. Cut the tubing to the required length.

- 3 Push the cut end of the tubing over the earhook, all the way up to the notch on the earhook.

- 4 Wear your Aqua+ with Mic Lock-Stirrup. You can leave Mic Lock-Stirrup attached to Aqua+.

Warnings

If your sound processor becomes unusually warm, remove it immediately and seek advice from your clinician. Parents/caregivers: if your recipient indicates discomfort, check if their sound processor is unusually warm.

Small parts (e.g. coil magnets) may be a choking hazard.

If the coil magnet is too strong or is in contact with the skin, pressure sores may develop at the coil site. If this happens, or if you experience any discomfort in this area, contact your clinician.

Do not apply continued pressure to the coil when in contact with the skin (e.g. sleeping while lying on coil, or using tight fitting headwear).

Store spare coil magnets safely and away from cards that may have a magnetic strip (e.g. credit cards, bus tickets, etc).

No modification of this equipment is allowed.

Use only as intended.

Report unexpected problems to your clinician.

Precautions

Always use Aqua+ with the blue Aqua+ coil, never with your usual coil.

When connecting the coil cable, always push the plug straight in; do not screw or twist the plug.

Where possible, avoid pressing the processor buttons with your fingernail as this can damage Aqua+.

If Aqua+ appears damaged or worn, replace it with a new one.

If your sound processor does get wet, dry it with a soft dry cloth and place it in your dry aid kit for at least eight hours.

Other Information

Environmental conditions

| Condition | Minimum | Maximum |
|---------------------------------|--------------|---------------|
| Storage & transport temperature | -10°C (14°F) | +55°C (131°F) |
| Operating temperature | +5°C (41°F) | +40°C (104°F) |
| Operating relative humidity | 0% RH | 90% RH |
| Operating pressure | 700 hPa | 1030 hPa |

Environmental protection

Your Aqua+ coil contains electronic components subject to the Directive 2002/96/EC on waste electrical and electronic equipment. Help protect the environment by not disposing of your Aqua+ coil with your unsorted household waste. Please recycle your Aqua+ coil according to your local regulations

Certification and applied standards

Your device fulfils the essential requirements listed in Annex 1 of the EC directive 90/385/EEC on Active Implantable Medical Devices as per the conformity assessment procedure in Annex 2.

The year in which authorisation to affix the CE mark was granted was 2015.

Symbols

The following symbols may appear on Aqua+ components and/or packaging:

Refer to instruction manual

Special care to be taken to ensure safety and effectiveness. Could cause damage to equipment

Potential safety hazards and serious adverse reactions. Could cause harm to person

Manufacturer

Authorised representative in the European Community

Catalogue number

Serial number

Batch code

Date of manufacture

Temperature limits

CE registration mark with notified body number

Rx Only

By prescription

Recyclable material

Dispose of electrical components in accordance with your local regulations

Warranty

Use of the Aqua+ will not impact your Nucleus Global Limited Warranty. Your Nucleus Global Limited Warranty will not be void in circumstances where Aqua+ is used with the Aqua+ coil in water with Nucleus CP810, CP910, and CP920 Sound Processors with rechargeable batteries only, in accordance with the Aqua+ instructions.

Legal statement

The statements made in this guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

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© Cochlear Limited 2015
are either trademarks or registered trademarks of Cochlear Limited.
**Cochlear Ltd** (ABN 96 002 618 073)
Cochlear Ltd (ABN 96 002 618 073)
**Cochlear Deutschland GmbH & Co. KG**