Cochlear™

Baha® 5 Power Sound Processor

User manual part A

English Svenska Dansk Suomi Norsk **ZONE 3**

3 13 23 33 43
GB SV DA FI NO

Hear now. And always

Sound processor type designations for models included in this User Manual are:

FCC ID: QZ3BAHA5POWER IC: 8039C-BAHA5POWER, IC model: Baha® 5 Power.

Statement:

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

(1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This device has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the device does cause harmful interference to radio or television reception, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from the one in which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.
- Changes or modifications can void the user's authority to operate the device.

Intended use

The Cochlear™ Baha® 5 Power uses bone conduction to transmit sounds to the cochlea (inner ear). It is indicated for people with conductive hearing loss, mixed hearing loss and single sided sensorineural deafness (SSD). Furthermore it is indicated for bilateral and paediatric recipients. Fitting range up to 55 dB SNHL. It works by combining a sound processor and a small titanium implant that is placed in the skull behind the ear. The skull bone integrates with the titanium implant through a process called osseointegration. This allows sound to

be conducted via the skull bone directly to the cochlea, which improves hearing performance. The sound processor can be used together with the Baha Softband.

The fitting is to be done either at a hospital, by an audiologist, or in some countries, by a hearing care professional.

List of countries:

Not all products are available in all markets. Product availability is subject to regulatory approval in the respective markets.

The products are in compliance with the following regulatory requirements:

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Equipment includes RF transmitter.

1

2

3

4

5

6

7

8

3

10


9

11

12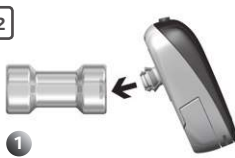

Welcome

Congratulations on your choice of the Cochlear™ Baha® 5 Power Sound Processor. You are now ready to use Cochlear's highly advanced bone conduction sound processor, featuring sophisticated signal processing and wireless technology.

This manual is full of tips and advice on how to best use and care for your Baha sound processor. By reading this manual and then keeping it handy for future reference, you will ensure that you get the most benefit out of your Baha sound processor.

Key to device *See figure* **1**

1. Visual indicator
2. Microphones
3. Battery compartment door
4. Tamper proof lock
5. Attachment point for the safety line
6. Volume rocker
7. Program button, Wireless audio streaming button
8. Plastic snap connector

Note on figures: The figures included on the cover correspond to information specific to this model of sound processor. Please reference the appropriate figure when reading. The images shown are not to scale.

Introduction

To ensure optimal performance, your hearing care professional will fit the sound processor to suit your needs. Be sure to discuss any questions or concerns that you may have regarding your hearing or use of this system with your hearing care professional.

Warranty

The warranty does not cover defects or damage arising from, associated with, or related to the use of this product with any non-Cochlear processing unit and/or any non-Cochlear implant. See the "Cochlear Baha Global Limited Warranty card" for more details.

Contacting Customer Service

We strive to provide you with the best possible products and services. Your views and experiences with our products and services are important to us. If you have any comments you would like to share, please contact us.

Customer Service – Cochlear Americas

13059 E. Peakview Avenue, Centennial
CO 80111, USA

Toll free (North America) 1800 523 5798

Tel: +1 303 790 9010, Fax: +1 303 792 9025

E-mail: customer@cochlear.com

Customer Service – Cochlear Europe

6 Dashwood Lang Road, Bourne Business Park,
Addlestone, Surrey KT15 2HJ, United Kingdom

Tel: +44 1932 26 3400, Fax: +44 1932 26 3426

E-mail: info@cochlear.co.uk

Customer Service – Cochlear Asia Pacific

1 University Avenue, Macquarie University,
NSW 2109, Australia

Toll free (Australia) 1800 620 929

Toll free (New Zealand) 0800 444 819

Tel: +61 2 9428 6555, Fax: +61 2 9428 6352

or Toll free Fax 1800 005 215

E-mail: customerservice@cochlear.com.au

Key to symbols

The following symbols will be used throughout this document.

Please refer to the list below for explanations:

	"Caution" or "Caution, consult accompanying documents"		ACMA symbol (Australian Communications and Media Authority) compliant
	Audible signal		Temperature limitation
	CE mark and Notified Body number		Interference risk
	Manufacturer	Year:	Year of Manufacture
	Batch Code		Serial Number
	Catalogue number	Rx Only	By prescription
	Made for iPod, iPhone, iPad		Bluetooth® Smart
	Refer to instructions/booklet. Note: Symbol is blue.		Product name and CE mark

On/off See figure 2

- 1 To turn on your sound processor close the battery door completely.
- 2 To turn off your sound processor gently open the battery door until you feel the first "click".

When your sound processor is turned off and then back on again, it will return to Program 1 and default volume level.

Status Indicator See figure 3

Visual indicators and beeps will alert you of changes to your sound processor. For a complete overview see the Visual indicators and beeps chart at the end of the section.

Change program/streaming

See figure 4

Your sound processor can be fitted with up to four programs suitable for different listening environments. The program button lets you choose from these pre-set programs and enable/disable wireless streaming.

- To change program press the program button located on the top of your sound processor.
- To activate audio wireless streaming press and hold the program button. Press the program button again to end wireless streaming and return to the previous program.

If you are a bilateral user, program changes you make to one device will automatically apply to the second device. This function can be enabled or disabled by your hearing care professional.

Note: You can also change program and adjust the volume using the optional Cochlear Baha Remote Control or Cochlear Wireless Phone Clip or directly from iPhone, iPad or iPod touch (see MFi for more information).

Adjust volume See figure 5

You can adjust the volume level of your sound processor using the volume rocker located on the side of the sound processor.

- To increase the volume, press the top of the volume rocker.
- To decrease the volume, press the bottom of the volume rocker.

Keylock

You can use the Keylock function to prevent inadvertent changes to your sound processor settings (such as program choice or volume level). This function can be enabled or disabled by your hearing care professional and is an important feature when the sound processor is being used by a child.

Wireless Accessories

Your sound processor is compatible with a range of Cochlear Wireless Accessories that can enhance your listening experience. Ask your hearing care professional to learn more about your options or visit www.cochlear.com.

Flight mode *See figure 6*

Wireless functionality must be deactivated when boarding a flight.

1. To turn on Flight mode first turn off the sound processor by opening the battery door.
2. Press and hold the program button and at the same time, close the battery door.

To turn off Flight mode simply turn the sound processor off and back on again.

Made for iPhone (MFi)

Your sound processor is a Made for iPhone (MFi) hearing device. This allows you to control your sound processor and stream audio directly from iPhone, iPad or iPod touch. For full compatibility details and more information visit www.cochlear.com.

1. To pair your sound processor turn on Bluetooth on your iPhone, iPad or iPod touch.
2. Turn off your sound processor and go to Settings > General > Accessibility on your iPhone, iPad or iPod touch.
3. Turn on your sound processor and select Hearing Aids in the Accessibility menu.
4. When displayed, tap on the sound processor name under "Devices" and press Pair when prompted.

Talk on the phone *See figure 7*

For best results, use a Cochlear Wireless Phone Clip or stream the conversation directly from your iPhone. When using a normal handheld phone, place the receiver near the microphone inlet of your sound processor instead of near your ear. Ensure the receiver does not touch the sound processor as this may result in feedback.

Replace the battery *See figure 8*

The visual indicators and beeps will alert you when there is approximately one hour of battery power remaining. At this time you may experience lower amplification. If the battery runs down completely the sound processor will stop working.

Use one of the batteries (zinc-air, non-rechargeable) included in the sound processor kit as a replacement. The batteries included in the kit reflect Cochlear's latest recommendations. Contact your hearing care professional for additional batteries.

1. To replace the battery hold the sound processor with the front facing up.
2. Gently open the battery door until it is completely open.
3. Remove the old battery and dispose of it according to local regulations.
4. Remove the new battery from the packet and peel away the sticker on the + side.
5. Insert the battery into the battery compartment with the + side facing up.
6. Gently close the battery door.

Tips

- To maximise battery life, switch off the sound processor when it is not in use.
- Battery life decreases as soon as the battery is exposed to air (when the plastic strip is removed) so be sure to only remove the plastic strip directly prior to use.
- Battery life depends on daily use, volume levels, wireless streaming, sound environment, program setting, and battery strength.
- If a battery leaks, replace it immediately.

Lock and unlock the battery door

See figure **9**

You can lock the battery door to prevent it from opening accidentally (tamper-proof). This is recommended especially when the sound processor is being used by a child.

- 1 To lock the battery door, close the battery door completely and place the Locking tool into the battery door slot. Slide the locking pin up into place.
- 2 To unlock the battery door, place the Locking tool into the battery door slot. Slide the locking pin down into place.

 **Caution: Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children and household pets. In the event a battery is accidentally swallowed, seek immediate medical attention at the nearest emergency centre.**

Attach the Safety line See figure **10**

To use the Safety line, simply attach it to the sound processor and clip it on to your shirt or jacket.

Cochlear recommends connecting the Safety line when engaging in physical activities. Children should use the Safety line at all times.

General care

Your sound processor is a delicate electronic device. Follow these guidelines to keep it in proper working order:

- Turn off and store the sound processor free from dust and dirt.
- Remove the battery when storing the sound processor for long periods.
- Avoid exposing your sound processor to extreme temperatures.
- Remove your sound processor before applying hair conditioners, mosquito repellent or similar products.
- Secure your sound processor with a Safety line during physical activities. If the physical activity involves contact, Cochlear recommends removing your sound processor.
- Your sound processor is not waterproof. Do not use it when swimming and avoid exposing it to heavy rain
- For cleaning your sound processor use the Baha sound processor Cleaning kit.

If the sound processor becomes very wet

1. Immediately open the battery door and remove the battery.
2. Put your sound processor in a container with drying capsules such as a Dri-Aid Kit. Leave it to dry out overnight. Drying kits are available from most hearing care professionals.

Feedback (whistling) problems

See figure **11**

- Check to ensure that your sound processor is not in contact with items such as glasses or a hat or in contact with your head or ear.
- Check that the battery compartment is closed.
- Check that there is no external damage to the sound processor.

Contact your hearing care professional if problems persist.

Share the experience


See figure **12**


Family members and friends can “share the experience” of bone conduction hearing using the test rod.

- 1 Turn on your sound processor and attach it on the test rod by tilting it into place.
- 2 Hold the test rod against the skull bone behind an ear. Plug both ears and listen.

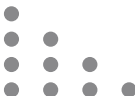Visual indicators

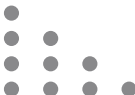Your hearing care professional can set up your processor to show the following light indications. All light flashes are orange.

General	
Light	What it means
 Steady light for 10 seconds	Start up.
 4 x dual flashes	Start up in Flight Mode.
 1-4 flashes	Change program. Number of flashes indicates the number of the current program.
 1 quick flash	Volume level increased/decreased by one step.
 1 long flash	Volume limit reached.
 Rapid flashes for 2.5 seconds	Low battery warning.

Wireless	
Light	What it means
 1 long flash followed by 1 short flash	Wireless streaming activated.
 1 long flash followed by 1 short flash	Change from one wireless accessory to another.
 1-4 flashes	End wireless streaming due to low battery voltage and return to program. Number of short flashes indicates the number of the current program.

Beeps

Your hearing care professional can set up your processor so you can hear the following beeps. The beeps are only audible to the recipient.

General	
Beeps	What it means
 10 beeps	Start up.
 10 x dual beeps	Start up in Flight Mode.
 1-4 beeps	Change program. Number of beeps indicates the number of the current program.
 1 beep	Volume level increased/decreased by one step.
 1 long beep	Volume limit reached.
 4 beeps 2 times	Low battery warning.

Wireless	
Beeps	What it means
 Ripple tone in upward melody	Wireless Accessory pairing confirmation.
 Ripple tone upward melody	Wireless streaming activated.
 2 x ripple tone downward melody	End wireless streaming due to low battery voltage and return to program.
 6 beeps followed by ripple tone upward melody (about 20 seconds after pairing)	MFi pairing confirmation.
 Ripple tone upward melody	Change from one wireless accessory to another

Warnings


General advice

A sound processor will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions.

- Infrequent use of a sound processor may not enable a user to attain full benefit from it.
- The use of a sound processor is only part of hearing rehabilitation and may need to be supplemented by auditory and lip reading training.

Warnings

- The sound processor is a digital, electrical, medical instrument designed for specific use. As such, due care and attention must be exercised by the user at all times.
- The sound processor is not waterproof!
- Never wear it in heavy rain, in the bath or shower!
- Do not expose the sound processor to extreme temperatures. It is designed to operate within the temperature ranges +5 °C (+41 °F) to +40 °C (+104 °F). In particular, battery performance deteriorates in temperatures under +5 °C.
- The processor should not be subjected, at any time, to temperatures below -20 °C (-4 °F) or above +50 °C (+122 °F).
- Contains small parts that may present a choking hazard.
- This product is not suitable for use in flammable and/or explosive environments.
- The sound processor and other external accessories should never be brought into a room with an MRI machine, as damage to the sound processor or the MRI equipment could occur.
- The sound processor must be removed before entering a room where an MRI scanner is located.
- If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.

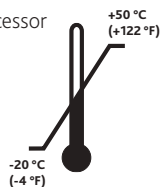- Portable and mobile RF (radio frequency) communications equipment can affect the performance of your sound processor.
- The sound processor is suitable for use in electromagnetic environments with power frequency magnetic fields of typical commercial or hospital levels. It is intended for use in an electromagnetic environment in which radiated RF (radio frequency) disturbances are controlled.

- Interference may occur in the vicinity of equipment with the symbol to the right.
- Dispose of batteries and electronic items in accordance with your local regulations.
- Discard your device as electronic waste according to local regulations.
- When the wireless function is activated, the sound processor uses low-powered digitally coded transmissions in order to communicate with other wireless devices. Although unlikely, nearby electronic devices may be affected. In that case, move the sound processor away from the affected electronic device.
- When using wireless functionality and the sound processor is affected by electromagnetic interference, move away from the cause of this interference.
- Be sure to deactivate wireless functionality when boarding flights.
- Turn off your wireless functionality by using the flight mode in areas where radio frequency emission is prohibited.
- Cochlear Baha wireless devices include an RF transmitter that operates in the range of 2.4 GHz–2.48 GHz.
- For wireless functionality, only use Cochlear Wireless accessories. For further guidance regarding e.g. pairing, please refer to the user guide of the relevant Cochlear Wireless accessory.
- No modification of this equipment is allowed.

Sound processor type designations for models included in this User Manual are:

FCC ID: QZ3BAHA5POWER IC: 8039C-BAHA5POWER, IC model: Baha® 5 Power.

Statement:

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This device has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the device does cause harmful interference to radio or television reception, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from the one in which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.
- Changes or modifications can void the user's authority to operate the device.

Intended use

The Cochlear™ Baha® 5 Power uses bone conduction to transmit sounds to the cochlea (inner ear). It is indicated for people with conductive hearing loss, mixed hearing loss and single sided sensorineural deafness (SSD). Furthermore it is indicated for bilateral and paediatric recipients. Fitting range up to 55 dB SNHL. It works by combining a sound processor and a small titanium implant that is placed in the skull behind the ear. The skull bone integrates with the titanium implant through a process called osseointegration. This allows sound to

be conducted via the skull bone directly to the cochlea, which improves hearing performance. The sound processor can be used together with the Baha Softband.

The fitting is to be done either at a hospital, by an audiologist, or in some countries, by a hearing care professional.

List of countries:

Not all products are available in all markets. Product availability is subject to regulatory approval in the respective markets.

The products are in compliance with the following regulatory requirements:

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Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey, KT15 2HJ, United Kingdom Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

Cochlear (HK) Limited Unit 1204, 12/F, 297-307 Hennessy Road, Wan Chai, Hong Kong Tel: +852 2530 5773 Fax: +852 2530 5183

Cochlear Medical Device (Beijing) Co Ltd Unit 2208-2212, Gemdale Tower B, 91 Jianguo Road, Chaoyang District, Beijing 100022, PR China

Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

Cochlear Korea Ltd 1st floor, Cheongwon Building 33, Teheran-ro 8 gil, Gangnam-gu, Seoul, Korea Tel: +82 2 533 4663 Fax: +82 2 533 8408

Cochlear Benelux NV Schaliënhoedreef 20 I, 2800 Mechelen, Belgium Tel: +32 1579 5511 Fax: +32 1579 5500

Cochlear Medical Device Company India PVT Ltd Platina Bldg, Ground Floor, Plot No. C 59, G Block, BKC, Bandra East, Mumbai 400051, India

Tel: +91 22 6112 1111 Fax: +91 22 61121100

Cochlear Italia SRL Via Larga 33, 40138 Bologna, Italy Tel: +39 051 601 53 11 Fax: +39 051 39 20 62

Cochlear France S.A.S. 135 route de Saint Simon, 31035 Toulouse, France Tel: +33 5 34 63 85 85 (international) Tel: 0805 200 016 (national)

Fax: +33 5 34 63 85 80

Cochlear Nordic AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden Tel: +46 31 335 14 61 Fax: +46 31 335 14 60

Cochlear Tibbi Cihazlar ve Sağlık Hizmetleri Ltd. Sti. Cubuklu Mah. Bogazici Cad. Bogazici Plaza No: 6/1 Kavacik, 34805 Beykoz-Istanbul, Turkey

Tel: +90 216 538 5900 Fax: +90 216 538 5919

Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

www.cochlear.com

Cochlear Baha 5 sound processors are compatible with iPhone 6s Plus, iPhone 6s, iPhone 6 Plus, iPhone 6, iPhone SE, iPhone 5s, iPhone 5c, iPhone 5, iPad Pro, iPad Air 2, iPad Air, iPad (4th generation), iPad mini 4, iPad mini 3, iPad mini 2, iPad mini with retina display, iPad mini and iPod touch (5th generation) running iOS 7.X or later.

"Made for iPhone" means that an electronic accessory has been designed to connect specifically to iPhone and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone may affect wireless performance.

Ardium, Baha, Baha SoftWear, BCDrive, DermaLock, EveryWear, Vistafix and Windshield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB. Cochlear, SmartSound, True Wireless, Hear now. And always and the elliptical logo are either trademarks or registered trademarks of Cochlear Limited. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Cochlear Limited is under license. iPhone, iPad and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.

© Cochlear Bone Anchored Solutions AB 2016. All rights reserved. NOV16.

0413

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Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland Tel: +41 61 205 8204 Fax: +41 61 205 8205