Cochlear™ Chargers

User Guide

EN DA FI NO SV LT LV ET

Hear now. And always

Contents

EN	Cochlear™ Chargers User Guide	1
DA	Cochlear™ Oplader brugervejledning	29
FI	Cochlear™-laturien käyttäjän opas	57
NO	Brukerveiledning for Cochlear™-ladere	85
SV	Bruksanvisning för Cochlear™-laddare	113
LT	„Cochlear™“ kroviklių naudotojo vadovas	141
LV	Cochlear™ lādētāju lietotāja rokasgrāmata	169
ET	Cochlear™ laadijate kasutusjuhend	197

Contents

About this guide	3
Charge your processor	4
Use the Home Charger	5
Use the Portable Charger	8
Check the charge level	10
Charge the Portable Charger.	11
Charge your processor	13
Regular care	16
Troubleshooting	17
Cautions	18
Warnings	19
Specifications	20
Other information.	24

Symbols used in this guide

NOTE
Important information or advice.

TIP
Time-saving hint.

CAUTION (no harm)
Special care to be taken to ensure safety and effectiveness.
Could cause damage to equipment.

WARNING (harmful)
Potential safety hazards and serious adverse reactions.
Could cause harm to person.

About this guide

This guide is intended for people who use the:

- Cochlear™ Home Charger
- Cochlear™ Portable Charger.

NOTES

- See the Cautions and Warnings sections for safety advice relating to the use of the chargers, accessories and components.
- Please also refer to your *Important Information* document for essential advice that applies to Cochlear implant systems.

Charge your processor

Your processor has an internal battery that needs regular charging.

To recharge your processor's internal battery you can:

- place it in the **Home Charger** (see page 5) or
- attach it to the **Portable Charger** (see page 8).

Please charge your sound processor as soon as you receive it.

WARNING

To charge your processor:

- Use Cochlear equipment only.
- Do not use non-Cochlear equipment.

CAUTION

To optimise battery performance, avoid charging your sound processor in high ambient temperatures.

Use the Home Charger

The Home Charger will store, dry and charge your processor.

To fully charge a depleted sound processor takes approximately 4½ hours.

Home Charger equipment includes:

- Home Charger unit
- Power adapter
- USB cable to connect the power adapter to the charger.

NOTE

For optimum performance, we recommend you connect the Home Charger to a power outlet with the power adapter provided.

To use the Home Charger to charge your processor:

1. **Plug** the USB cable connector into the power adapter.

2. **Plug** the other end of the cable into the power connector on the back of the Home Charger.

3. **Plug** the power adapter into a power outlet and **switch** the power on.

When power is on, the Home Charger power light displays **steady green**.

4. **Open** the lid and **place** your sound processor inside.

CAUTION

Do not press down on the processor.

NOTE

Safety Lines and SoftWear Pads can remain attached to the processor while in the Home Charger.

5. **Close** the lid.
As it begins to charge, your processor light **pulses green**.

6. When fully charged, the processor light displays **steady green**.
Open the lid and remove the processor.

INDICATOR LIGHTS WHAT IT MEANS

Orange flashes

Problem with power or temperature.

Steady orange

Charger is faulty. Contact your clinician to arrange a replacement charger.

Use the Portable Charger

You can carry the Portable Charger* and use it as needed to charge your processor. The Portable Charger contains an internal battery that requires charging before use.

The Portable Charger takes approximately 3 hours to fully charge. It can be connected to the back of the Home Charger for charging overnight.

When fully charged, the Portable Charger provides one full charge to your processor. To fully charge a depleted sound processor takes approximately 3–4 hours.

Portable Charger equipment includes:

- Portable Charger unit
- USB cable for charging
- Charger cable for connecting to your processor.

**WARNING**

- Remove the Portable Charger immediately if it becomes hot and contact your clinician.
- Do not place the Portable Charger under clothing of children or anyone who is unable to notify you if the charger becomes hot.

**CAUTION**

To optimise battery life:

- Fully charge the Portable Charger immediately, even if you don't plan to use it.
- Every 6 months, fully charge the Portable Charger.

Check the charge level


To check the charge level, **press the button** on the front of the Portable Charger.

Green lights display to indicate the available charge.

LIGHTS DISPLAYED

WHAT IT MEANS

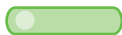1 light – ¼ full

2 lights – ½ full

3 lights – ¾ full

4 lights – fully charged

Green lights

Charge the Portable Charger

1. **Plug** the small USB-C cable connector into the Portable Charger.

2. **Plug** the larger USB-A cable connector into a power source. Options include:

- Home Charger DC 5V OUT socket
- Home Charger power adapter
- Laptop USB port*


*USB ports must be high power USB 2.0 or higher. If you use a USB hub to connect more than one USB device to a port, we recommend you use a powered hub.

3. After approximately 3 hours, **press** the button to check the charge level. When fully charged, **4 lights** will display.
4. When fully charged, **unplug** the Portable Charger.

It is now ready to take with you in case you need to recharge your processor.

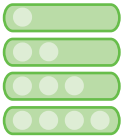Make sure you also take the Portable Charger cable so you can connect the charger to the processor.

INDICATOR LIGHTS	WHAT IT MEANS
 Green flashes	Power is connected for charging. Number of lights = amount of charge. Light display stops 30 seconds after charger is connected to power.
 Orange flashes	Problem with power or temperature.
 Steady orange	Charger is faulty. Contact your clinician to arrange a replacement charger.

Charge your processor

To use the Portable Charger to charge your processor:

1. **Connect** the cable to the **Portable Charger**.

The grey plug corresponds to the grey socket.

2. If the socket cover is in place, **remove** the socket cover from the back of your processor.

3. **Connect** the other end of the cable to your **processor**.

As it begins to charge, the processor light **pulses green**.

Continued on next page ...

4. **Wear** the charger by attaching to clothing (e.g. pocket or collar) or on a lanyard around your neck.

Use the hole on the clip to connect a **lanyard**.

5. **Place** the processor on your head.

6. When fully charged, the processor light displays **steady green**.

Disconnect the cable by lifting the connector upwards.

7. **Store** the charger and cable carefully to avoid damage. If removing the cable, hold the connector on either side and pull.

**NOTE**

Two cable lengths are available for the Portable Charger: 30 cm and 50 cm.

Regular care

CAUTION

- Do not use cleaning agents or alcohol to clean the chargers.
- Turn your processor off before cleaning or performing maintenance.

Every day

Check your chargers for dirt and moisture. Wipe the chargers with a soft dry cloth.

Store your chargers, power adapter and cables in a safe place to avoid damage.

Every six months

Charge the Portable Charger to optimise battery life.

Troubleshooting

An **orange light** (error) will flash on the chargers if they are:

- not receiving enough power, or
- used in temperatures outside their operating range.

What should you do?

- Check that cables are connected properly.
- Try using the charger in a different location. The temperature might be outside the charger's operating range.
- If using a USB port, try another port. The type of USB port could be incorrect.
- If using a USB hub:
 - Check you are using a powered hub. The type of hub could be incorrect.
 - Remove some devices. The hub may have too many devices connected.

Cautions

To charge your processor:

- Use Cochlear equipment only.
- Do not use non-Cochlear equipment.

Warnings

- Removable parts of the system can be lost or may be a choking or strangulation hazard. Keep out of reach of children.
- Unsupervised use of long cables (e.g. Portable Charger cable) may present a risk of strangulation.
- Do not place the device or accessories inside any part of your body (e.g. nose, mouth).
- Carers must routinely check devices that are worn on the body for signs of overheating (e.g. Portable Charger). Remove the device immediately if it becomes hot and contact your clinician.
- Do not place the chargers or parts in any household devices (e.g. microwave oven, dishwasher, dryer).
- Do not expose the chargers or parts to heat (e.g. never leave them in sunlight, behind a window or in a car).
- Your chargers and other parts of the system contain complex electronic parts. These parts are durable but must be treated with care.
- No modification of this equipment is allowed. Warranty will be void if modified.

Specifications

Home Charger specifications

The Home Charger is the primary charging unit and includes capability to dry your processor. The Home Charger comprises:

- Wireless charging coil, internal fan and custom electronics
- Power input and error light indicators
- Built-in temperature sensor
- Power adaptor
- USB cable

MATERIALS

- | | |
|-------------------|--|
| External housings | <ul style="list-style-type: none"> • Polycarbonate / acrylonitrile butadiene styrene • Polycarbonate • Silicone |
|-------------------|--|

DIMENSIONS

Home charger unit
(typical values)

HEIGHT

42 mm

WIDTH

85 mm

DEPTH

83 mm

WEIGHT

Charging unit (typical value)

WEIGHT

111 g

OPERATING CHARACTERISTICS	VALUE/RANGE
Operating frequency (inductive charging link)	100 kHz to 160 kHz
Input operating voltage	4.75 V to 5.25 V DC
Output voltage	4.75 V to 5.25 V DC
Output current	200 mA
Power consumption	3.5 W

COMPATIBLE PRODUCTS

The Home Charger can be used to charge the:

- Kanso 2 Sound Processor (model number: CP1150)
- Cochlear CP1150S Surgical Processor.

Portable Charger specifications

The Portable Charger's internal battery provides charge to the sound processor battery via the charger cable. The Portable Charger comprises:

- Custom electronics with internal battery
- Battery charge level check button and light indicators
- Built-in temperature sensor
- Charger cable to connect to sound processor
- USB cable

MATERIALS

External housings	Polyamide
-------------------	-----------

DIMENSIONS

DIMENSIONS	LENGTH	WIDTH	HEIGHT
Portable charger unit (typical values)	59.5 mm	28 mm	13.7 mm (excludes clip)

WEIGHT

Portable charger unit and cable (typical value)	22 g
---	------

WEIGHT

COMPATIBLE PRODUCTS

The Portable Charger can be used to charge the Kanso 2 Sound Processor (model number: CP1150).

OPERATING CHARACTERISTICS	VALUE/RANGE
Input operating voltage	4.75 V to 5.25 V DC
Output voltage	4.90 V to 5.25 V DC
Output current	120 mA
Power consumption	1 W
Button functions	Check battery level
Battery type	Lithium ion
Battery capacity	1.18 Wh
Ingress Protection Rating when connected to the Kanso 2 Sound Processor	IP54

Environmental conditions

CONDITIONS	MINIMUM	MAXIMUM
Storage and transport temperature	-10° C (14° F)	+55° C (131° F)
Storage and transport humidity	0% RH	90% RH
Operating temperatures	+5° C (41° F)	+40° C (104° F)
Operating relative humidity	0% RH	90% RH
Operating pressure	700 hPa	1060 hPa

Other information

Electromagnetic compatibility (EMC)

WARNING

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 in.) to any part of the Cochlear Home Charger or Cochlear Portable Charger, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Interference may occur in the vicinity of equipment marked with this symbol:

Environmental protection

Your chargers contain electronic components subject to the Directive 2002/96/EC on waste electrical and electronic equipment.

Help protect the environment by not disposing of your chargers with your unsorted household waste. Please recycle according to your local regulations.

Equipment classification

Your chargers are internally powered equipment Type B applied part as described in the international standard IEC 60601-1:2005/A1:2012, Medical Electrical Equipment – Part 1: General Requirements for Basic Safety and Essential Performance.

Labelling symbols

The following symbols may appear on your charger components and/or packaging:

Refer to instruction manual

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

M/N

Model number

Authorised representative in the European Community

Catalogue number

Serial number

Batch code

Date of manufacture

Temperature limits

CE registration mark

Ingress Protection Rating

IP21

- Protected against solid objects over 12.5 mm e.g. fingers or similar objects
- Protected against dripping water (vertically falling drops)

Rx Only By prescription

Recyclable material

Dispose of electrical components in accordance with your local regulations

MD

Medical device

CP1150 Kanso 2 Sound Processor (model number CP1150)

CP1150S Cochlear CP1150S Surgical Processor

Serious incidents

Whilst serious incidents in relation to medical devices are rare, it is acknowledged that incidents may happen. As an organisation, Cochlear recognises the potential for harm and will respond to any reported serious incident.

What is a serious incident?

A 'serious incident' means any event that directly or indirectly has caused or could have caused an unexpected or unwanted event including any of the following:

- The death of a patient, user or other person,
- The temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- A serious public health threat.

Reporting a serious incident

There is no definitive list of events/incidents that constitute a serious incident, however all serious incidents should be reported to:

- your local Cochlear office
www.cochlear.com/intl/contact/global-offices
- your National Competent Authority
http://ec.europa.eu/growth/sectors/medical-devices/contacts_en

Privacy and the collection of personal information

During the process of receiving a Cochlear device, personal information about the user/recipient or their parent, guardian, carer and hearing health professional will be collected for use by Cochlear and others involved in care with regard to the device.

For more information please read Cochlear's Privacy Policy on www.cochlear.com or request a copy from Cochlear at the address nearest you.

Legal statement

The statements made in this guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

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Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada
Tel: +1 416 972 5082 Fax: +1 416 972 5083

Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland
Tel: +41 61 205 8204 Fax: +41 61 205 8205

Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom
Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

Cochlear Benelux NV Schaliënhoedreef 20 i, B-2800 Mechelen, Belgium
Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

Cochlear France S.A.S. 135 Route de Saint-Simon, 31035 Toulouse, France
Tel: +33 5 34 63 85 85 (International) or 0805 200 016 (National) Fax: +33 5 34 63 85 80

Cochlear Italia S.r.l. Via Larga 33, 40138 Bologna, Italy
Tel: +39 051 601 53 11 Fax: +39 051 39 20 62

Cochlear Nordic AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden
Tel: +46 31 335 14 61 Fax: +46 31 335 14 60

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Cochlear (HK) Limited Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong
Tel: +852 2530 5773 Fax: +852 2530 5183

Cochlear Korea Ltd 1st floor, Cheongwon Building 33, Teheran-ro 8 gil, Gangnam-gu, Seoul, Korea
Tel: +82 2 533 4450 Fax: +82 2 533 8408

Cochlear Medical Device (Beijing) Co., Ltd
Unit 2608-2617, 26th Floor, No.9 Building, No.91 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

Cochlear Medical Device Company India Pvt. Ltd.
Ground Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

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Cochlear Middle East FZ-LLC
Dubai Healthcare City, Al Razi Building 64, Block A, Ground Floor, Offices IR1 and IR2, Dubai, United Arab Emirates
Tel: +971 4 818 4400 Fax: +971 4 361 8925

Cochlear Latinoamérica S.A.
International Business Park, Building 3835, Office 403, Panama Pacifico, Panama
Tel: +507 830 6220 Fax: +507 830 6218

Cochlear NZ Limited
Level 4, Takapuna Towers, 19-21 Como St, Takapuna, Auckland 0622, New Zealand
Tel: + 64 9 914 1983 Fax: 0800 886 036

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ACE, Advance Off-Stylet, AOS, AutoNRT, Autosensitivity, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Codacs, Contour, Contour Advance, Custom Sound, ESPrit, Freedom, Hear now. And always, Hugfit, Hybrid, Invisible Hearing, Kanso, MET, MicroDrive, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Outcome Focused Fitting, Off-Stylet, Slimline, SmartSound, Softip, SPrint, True Wireless, the elliptical logo, and Whisper are either trademarks or registered trademarks of Cochlear Limited. Ardium, Baha, Baha SoftWear, BCDrive, DermaLock, EveryWear, SoundArc, Vistafix, and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB.

© Cochlear Limited 2020

D1704322 ISS1 APR20

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Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.